



High-Level Instrument Disinfectant
Broad-Spectrum Antimicrobial

gigasept™ OPA

Our Plus - Your benefits

- Effective against bacteria, fungi, viruses and spores
- Effective against GTA-resistant strains of Mycobacteria
- Convenient to use as no activation is required
- Fast & Effective
- Excellent material compatibility
- Non-corrosive solution

Applications / Direction for use

See direction for use before using the product or any kind of application

High Level Disinfection

gigasept™ OPA can be used or reused as a high-level disinfectant for a maximum of 14 days at 25° C with an immersion time of at least 5 minutes

Warning and Precaution

gigasept™ OPA should not be used to reprocess any Urological Instruments, which have been used in the treatment of patient with bladder cancer. Use only in ventilated areas, wear suitable protective clothing, gloves and eye / face protection. In case of contact with eyes, rinse immediately with plenty of water and seek medical advice. May cause sensitization by inhalation & skin contact.

Product Data

Composition

Ortho Phthalaldehyde 0.55%

Delivery forms / Packaging units

For any information please contact our distributor.

Container sizes	Pack Units	Art no.
5L Bottle	2 bottles per carton	70001125

Environmental information

schülke products are manufactured with advanced, safe and environmentally friendly production processes while always maintaining high quality standards.

Expert opinion and information

For an overview of our products containing gigasept™ OPA, or any individual query please contact our schülke representative or write to us.

Email: customer care.India@schuelke.com

High Level Disinfection of instruments provides a crucial protection to patients



To keep your patients safe from any instrument-associated infection

INTRODUCING

gigasept™ OPA

Professional reprocessing standard for your instruments



Schülke & Mayr GmbH is certified according to DIN EN ISO 9001, DIN EN ISO 14001 and DIN EN ISO 13485 (Reg.-No. 004567-MP23) and has a validated environmental system in accordance with the Eco Audit Regulation (Reg.-No. DE-150-00003).

Schülke & Mayr GmbH
22840 Norderstedt, Germany
Phone +49 (0) 40 - 521 00 - 0
Fax +49 (0) 40 - 521 00 - 318
www.schuelke.com
mail@schuelke.com

Schulke India Pvt. Ltd.
A-24/9, Mohan Co-op. Ind. Est.,
Mathura Road, New Delhi-110044.
Phone +91 - 11 - 4055 0200
Fax +91 - 11 - 4055 0201/02
customer care.india@schuelke.com