



Eco-Friendly Surface and Environment Disinfectant

## mikrozid™ HP-10

### Our Plus - Your benefits

- Tested to meet the highest quality standards
- Effective against bacteria, fungi, viruses, spores
- Fast & effective
- Non-Toxic and Non-Irritating

### Applications / Direction for use

#### For Critical Area Fumigation and Surface Disinfection

##### Area of usage

Hospitals, OT's, ICU's, Polyclinics, Dental clinics, OPD's, Blood Banks, Clinical/Diagnostic laboratories

##### Direction for Use

- Surface disinfection
- 10% v/v in DM water with a contact time of 60 Min.

##### Fumigation

- 20% v/v solution in DM water (200 ml of mikrozid HP-10 in 800 ml of water)
- 1 L/1000 cu-ft for areal disinfection in 60 Min. For best results use ULV fogger for dispensing

### Precaution & Contraindications

- Store in a cool & dry place
- Do not expose to sunlight
- The concentrated product may cause skin irritation. in case of accidental spillage or exposure, wash with plenty of water
- Keep away from children
- Do not ingest

### Product Data

#### Composition

- Stabilized Formulation of Hydrogen Peroxide 10% v/v
- Silver 0.01% w/v

Under Technical Collaboration with Sanosil AG

### Delivery forms / Packaging units

For any information please contact our distributor.

Container sizes	Pack Units	Art no.
1L Bottle	12 bottles per carton	70001126

### Environmental information

schülke products are manufactured with advanced, safe and environmentally friendly production processes while always maintaining high quality standards.

### Expert opinion and information

For an overview of our products containing mikrozid™ HP-10, or any individual query please contact our schülke representative or write to us.

Email: [customer care.india@schuelke.com](mailto:customer care.india@schuelke.com)



Schülke & Mayr GmbH is certified according to DIN EN ISO 9001, DIN EN ISO 14001 and DIN EN ISO 13485 (Reg.-No. 004567-MP23) and has a validated environmental system in accordance with the Eco Audit Regulation (Reg.-No. DE-150-00003).

Schülke & Mayr GmbH  
22840 Norderstedt, Germany  
Phone +49 (0) 40 - 521 00 - 0  
Fax +49 (0) 40 - 521 00 - 318  
[www.schuelke.com](http://www.schuelke.com)  
[mail@schuelke.com](mailto:mail@schuelke.com)

Schulke India Pvt. Ltd.  
A-24/9, Mohan Co-op. Ind. Est.,  
Mathura Road, New Delhi-110044.  
Phone +91 - 11 - 42595050 / 30796000  
Fax +91 - 11 - 4055 0201/02  
[customer care.india@schuelke.com](mailto:customer care.india@schuelke.com)